

# OraQuick HIV Rapid Test

For use with oral fluids

Store Kits: 2 - 30° C

- Check kit before use. Use only items that have not expired or been damaged.
- Bring kit and previously stored specimens to room temperature prior to use.
- Always use universal safety precautions when handling specimens. Keep work areas clean and organized.

This outline is not intended to replace the product insert or your standard operating procedure (SOP).




1. Collect test items and other necessary lab supplies.



2. Set reusable stand on a flat, level surface. Partially remove device from package and label device and developer vial with client identification number.



3. Carefully uncap the developer vial and place vial into the stand.

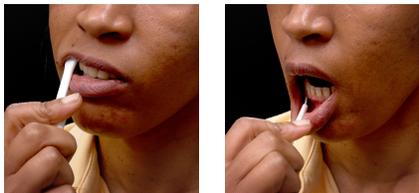

4. Instruct the client to use the pad end of the test device to swab completely across the outside of the upper and lower gums, one time around.



5. Insert the device pad completely into the vial with the result window facing forward.



6. Wait 20 minutes (no longer than 40 min.) before reading the results.




7. Read and record the results and other pertinent info on the worksheet.

## OraQuick HIV Rapid Test Results

### Reactive

2 lines of any intensity appear in both the control and test areas.

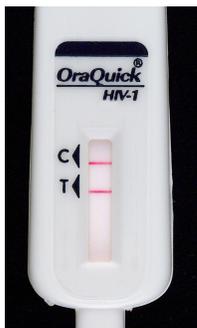

### Non-reactive

1 line appears in the control area and no line in the test area.



### Invalid

No line appears in the control area. Do not report invalid results. Repeat test with a new test device even if a line appears in the test area.



Use of trade names and commercial sources is for identification only and does not imply endorsement by WHO, the Public Health Service, or by the U.S. Department of Health and Human Services (2005).

